

User Manual

CareSens™ S

Fit BT

Blood Glucose
Monitoring System



Easy Data Transfer by
Bluetooth Connection



Indicate Blood Glucose
Range with Intuitive
Symbols

KET

Ketone Testing Alarm



Welcome to the CareSens S Fit BT Blood Glucose Monitoring System

Thank you for choosing the CareSens S Fit BT Blood Glucose Monitoring System. The system provides you with rapid and convenient blood glucose *in vitro* (i.e., outside the body) diagnostic monitoring. Test results can be sent to a smartphone through Bluetooth® wireless connection.

Bluetooth® wireless technology

Bluetooth wireless technology is used by some smartphones and many other devices. CareSens S Fit BT meter uses Bluetooth wireless technology to pair* and to send your glucose results to a smartphone.

The CareSens S Fit BT meter is designed to work with the SmartLog mobile app. When using the CareSens S Fit BT System, we suggest you pair your CareSens S Fit BT meter with your smartphone and track your results.

* The process of creating a connection between two Bluetooth devices. An auto-generated passkey has to be exchanged between the two devices. Once the devices are paired, they will automatically communicate with each other when the Bluetooth feature is activated.

- No part of this document may be reproduced in any form or by any means without the prior written consent of i-SENS.
- The information in this manual is correct at the time of printing. However, i-SENS reserves the right to make any necessary changes at any time without notice as our policy is one of continuous improvement.

Warning

Turn off the Bluetooth feature in areas where the use of wireless devices is restricted, such as hospitals, some healthcare professional offices, and airplanes.

Trademarks

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by i-SENS, Inc. is under license. All other trademarks and trade names are those of their respective owners.

Note

The SmartLog mobile app may not be compatible with all smartphones.

Table of Contents

Information

Important Information: Read This First!	6
Specifications.....	9
CareSens S Fit BT Blood Glucose Monitoring System.....	11
Inserting or Replacing the Battery.....	12
Caring for Your System.....	14
CareSens S Blood Glucose Test Strip	16
CareSens S Fit BT Blood Glucose Meter.....	18
CareSens S Fit BT Blood Glucose Meter Display	19

Preparation

Setting Up Your System	21
Setting the Strip Expiration Date Indicator.....	32
Checking the System.....	33

Testing

Using the Lancing Device	38
HI and Lo Messages.....	47
Target Blood Glucose Ranges.....	48

Additional Functions

Meter Memory.....	50
Setting the Alarm Function.....	53

Maintenance

Understanding Error Messages.....	56
General Troubleshooting	58
Performance Characteristics.....	59
Warranty Information.....	63

Important Information: Read This First!




For optimum safety and benefits, please read the entire manual contents before using the system.

Intended use:

CareSens S Fit BT Blood Glucose Monitoring System is intended for the quantitative measurement of glucose in fresh capillary whole blood from the fingertip.

The CareSens S Fit BT Blood Glucose Monitoring System is intended for use outside the body (*in vitro* diagnostic use) and is intended for use as an aid to monitor the effectiveness of diabetes management. The system is for self-testing or healthcare professional use and should not be used for the diagnosis of or screening for diabetes or for testing newborns.

Meaning of symbols used:

	CE Mark
	Authorised representative in the European Community/ European Union
	<i>In vitro</i> diagnostic medical device
	Caution
	Biological risks
	Do not re-use
	Consult instructions for use
	WEEE (waste electrical and electronic equipment)
	Temperature limit
	Manufacturer
	Batch code
	Serial number
	Use-by date

Specifications

- Glucose in blood samples reacts with the chemical in the test strip to produce a small electrical current. The CareSens S Fit BT meter detects this electrical current and measures the amount of glucose in the blood sample.
- The CareSens S Fit BT blood glucose meter is designed to minimise code related errors in monitoring by using the no-coding function.
- The CareSens S Fit BT blood glucose meter should be used only with the CareSens S Test Strips.
- An abnormally high or low red blood cell count (hematocrit level over 65 % or below 15 %) may produce inaccurate results.
- If your test result is below 3.3 mmol/L or above 13.3 mmol/L, consult a healthcare professional immediately.
- Inaccurate results may occur in severely hypotensive individuals or patients in shock. Inaccurate low results may occur for individuals experiencing a hyperglycemic hyperosmolar state, with or without ketosis. Critically ill patients should not be tested with blood glucose meters.
- Inaccurate results may occur in patients undergoing oxygen therapy.

If you need assistance, please contact the Spirit Health 24/7 Freephone Support Line at 0800 881 5423 or go to www.spirit-health.com.

Product specifications

Measurement range	1.1–33.3 mmol/L
Sample size	Minimum 0.5 µL
Test time	6 seconds
Sample type	Fresh capillary whole blood
Calibration	Plasma-equivalent
Assay method	Electrochemical
Battery life	1,000 tests
Power	One 3.0 V lithium battery (disposable, type CR2032)
Memory	1,000 test results
Size	95 x 49 x 17.5 mm
Weight	51.5 g (with battery)
Bluetooth technology	<ul style="list-style-type: none">• Frequency range: 2.4–2.4835 GHz• Operating range distance: 10 meters (unobstructed)• Operating channels: 40 channels• Security encryption: 128-bit AES (Advanced Encryption Standard)

CareSens S Fit BT Blood Glucose Monitoring System

Operating ranges

Temperature	5–45 °C
Relative humidity	10–90 %
Hematocrit	15–65 %

Storage/Transport conditions

Temperature	Meter (with battery)	0–50 °C
	Test strip	1–30 °C
	Control solution	8–30 °C
Relative humidity	Test strip	10–90 %

CareSens S Fit BT Blood Glucose Monitoring System includes the following items:

- * CareSens S Fit BT Blood Glucose Meter
- * Instructions for Use

CareSens S Fit BT Blood Glucose Monitoring System may include the following items:

- * CareSens S Blood Glucose Test Strips
 - * Battery
 - * Lancets
 - * Lancing Device
 - * Logbook
 - * Carrying Case
-
- Check all the components after opening the CareSens S Fit BT Blood Glucose Monitoring System package. The exact contents are listed on the main box.
 - Please contact the Spirit Health 24/7 Freephone Support Line for a free data transmission cable at 0800 881 5423 or go to www.spirit-health.com.


Inserting or Replacing the Battery

The CareSens S Fit BT meter uses one 3.0 V lithium battery. Before using the meter, check the battery compartment and insert a battery if empty.

When the **+** **-** symbol appears on the display while the meter is in use, the battery should be replaced as soon as possible. The test results may not be saved if the battery runs out completely.

Step 1

Make sure the meter is turned off. Push the cover in the direction of the arrow to open the battery compartment.

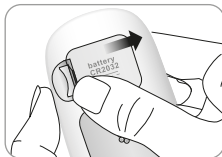

Step 2

Remove the used battery. Slip your thumb finger and pull it out with your index and thumb fingers as shown in the diagram. Insert a new battery with the + side facing up and make sure the battery is inserted firmly.



Step 3

Place the cover on the battery compartment. Push it down until you hear the tab click into place.

Note

Removing the meter battery will not affect your stored results. However, you may need to reset your meter settings. See page 21.

Caring for Your System

Use a soft cloth or tissue to wipe the meter exterior. If necessary, dip the soft cloth or tissue in a small amount of alcohol.

Do not use organic solvents such as benzene or acetone, or household and industrial cleaners that may cause irreparable damage to the meter.

Caution


- Do not expose the meter to direct sunlight, heat, or excessive humidity for an extended period of time. It is recommended to store and use the test system indoors.
- Do not let dirt, dust, blood, or water enter into the meter's test strip port.
- Do not drop the meter or submit it to strong shock.
- Do not try to fix or alter the meter in any way.
- Keep the meter away from strong electromagnetic field sources, such as cell phones and microwave ovens.
- Store all the meter components in the carrying case to prevent loss and help keep the meter clean.
- Avoid getting any liquid or moisture in the test strip vial. This can affect the test strips and cause inaccurate test results.
- Do not apply samples other than capillary whole blood or control solution to the test strip.
- Store the meter in a cool and dry place between 0–50 °C.

Disposal of the meter

If you need to throw your meter away, you should follow existing policies and procedures of your own country or region. For information about correct disposal, please contact your local council or authority. If you need assistance, contact the Spirit Health 24/7 Freephone Support Line at 0800 881 5423 or go to www.spirit-health.com.

CareSens S Blood Glucose Test Strip

The CareSens S Fit BT Blood Glucose Monitoring System measures blood glucose quickly and accurately. It automatically absorbs the small blood sample applied to the narrow edge of the strip.

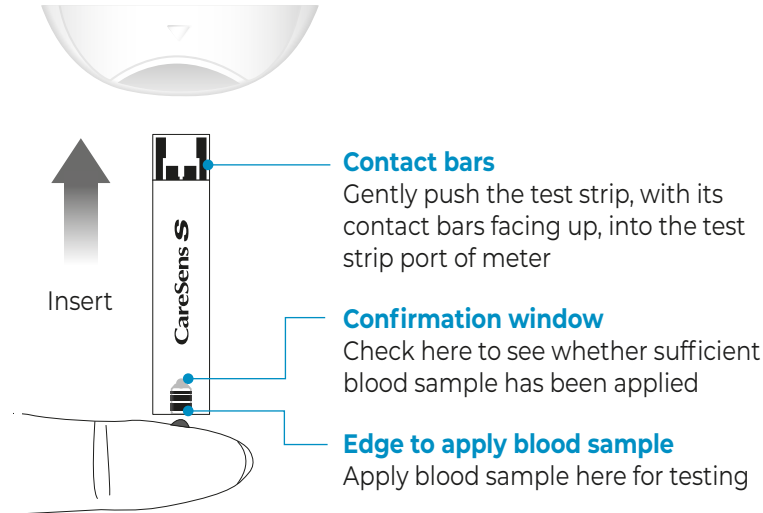

Warning

- The CareSens S test strips should be used only with fresh capillary whole blood samples.
- Do not reuse test strips.
- Do not use test strips past the expiration date.
- Test strips in new, unopened vials and test strips in vials that have been opened can be used up until the expiration date printed on the test strip box and vial label if the test strips are used and stored according to its storage and handling methods.
- Store test strips in a cool and dry place at a temperature between 1–30 °C and 10–90 % relative humidity.
- Keep test strips away from direct sunlight or heat and do not freeze.
- Store test strips only in their original vial.
- Close the vial tightly after taking out a test strip for testing and use the strip immediately.
- Avoid getting any liquid or moisture in the test strip vial. This can affect the test strips and cause inaccurate test results.
- Do not apply samples other than capillary whole blood or control solution to the test strip.
- Handle test strips only with clean and dry hands.
- Do not bend, cut, or alter test strips in any way.
- For detailed storage and usage information, refer to the CareSens S test strip package insert.
- Keep the meter and testing supplies away from young children.
- Drying agents in the vial cap may be harmful if inhaled or swallowed and may cause skin or eye irritation.

CareSens S Fit BT Blood Glucose Meter

Data Port

Used to transfer data from the meter to a computer with a cable

Display

Shows results and messages

Button

Turns the meter on, selects or changes information

S Button

Turns the meter on/off, confirms menu selections, and changes information

Test Strip Port


Insert test strip here



Note

- The cable for data management software can be ordered separately. Please contact your authorised i-SENS sales representative.
- The unit of measurement is fixed and it cannot be changed by the user.

CareSens S Fit BT Blood Glucose Meter Display

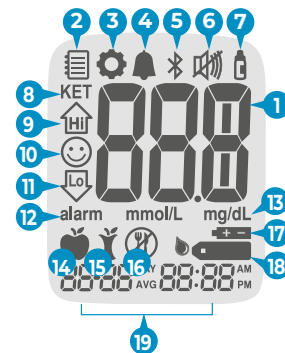

1	Test results	test results displaying panel
2	Memory symbol	appears when test results stored in the memory are displayed
3	Setting symbol	appears when in SET mode
4	PP2 alarm	appears when the post-meal alarm has been set
5	Bluetooth Symbol	appears when Bluetooth feature is on
6	Mute symbol	appears only when the sound is set to OFF
7	Control Solution flag	appears when the control solution test results are saved or displayed
8	KET symbol	appears when the test result is greater than 13.3 mmol/L
9	Hi	appears when the test result is greater than the selected hyperglycemia level

10 Smile symbol	appears when the test result is within the selected normal blood glucose range
11 Lo	appears when the test result is lower than the selected hypoglycemia level
12 alarm	appears when the time alarm has been set
13 mmol/L, mg/dL	unit for measuring blood glucose
14 Pre-meal test flag	used for tests done before eating
15 Post-meal test flag	used for tests done after eating
16 Fasting test flag	used for tests done after fasting for at least 8 hours
17 Battery symbol	indicates meter battery is running low and needs to be replaced
18 Blood insertion symbol	indicates meter is ready for the application of a drop of blood or control solution
19 Month/Day/Hour/Minute	appears date and time

Note

It is recommended to check if the display screen on the meter matches the illustration above every time the meter turns on. Do not use the meter if the display screen does not exactly match the illustration as the meter may show incorrect results.

Setting Up Your System


Press and hold the **S** button for 3 seconds to enter SET mode. After all settings are finished, press and hold the **S** button for 3 seconds to turn off the meter.

Press the ◀ or ▶ button to change values. Press and hold the ▶ button to scroll faster.

Step 1 Entering SET Mode

Press and hold the **S** button for 3 seconds to enter SET mode. After all the segments flash across the screen, 'SET' will show up.

Press the ◀ or ▶ button to select 'YES' and press the **S** button to go to the next step.

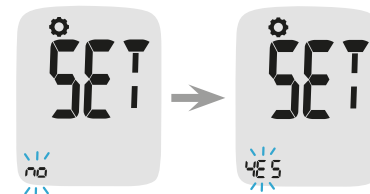

If you do not want to connect your meter to your smartphone, press the **S** button when the screen display 'bT' while 'OFF' is blinking on the bottom of the screen. Then, the meter will go to Step 4 Adjusting the Date and Time mode.

Note

Follow steps 2–3 to pair your meter and smartphone. Pairing allows the meter to communicate wirelessly with your smartphone.

Ensure that devices are within the maximum Bluetooth range (10 metres). Before pairing your meter and smartphone, download and install the SmartLog mobile app on your smartphone.

Setting Up Bluetooth


Step 2 Bluetooth Pairing

- 1 Smartphone:** If you want to pair (connect) your meter to your smartphone, launch the SmartLog mobile app and find the Accessories menu on your smartphone.
- 2 Smartphone:** Select the CareSens S Fit model from the meters list, and then tap **Bluetooth Register > Next**.
- 3 Meter:** With your meter turned off, press and hold the ► button for three seconds to enter pairing mode. The Bluetooth symbol and 'bT' will appear and 'YES' will blink at the bottom of the screen. Press the **S** button to select 'YES'.
- 4 Smartphone/Meter:** Tap **1** on your smartphone to set the Bluetooth pairing type setting while Bluetooth symbol and 'tYPE 1' are displayed on the meter's screen.



- 5 Smartphone/Meter:** Follow the on-screen instructions on SmartLog mobile app to begin searching your meter. Look for 'CareSens' and the last 4 characters of the meter serial number on the SmartLog mobile app screen to correctly identify your meter. Touch your meter's ID (CareSens XXXX) on the SmartLog mobile app screen.

Serial number

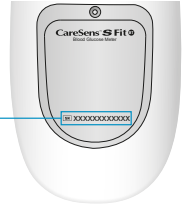

Note

Some content or menus may differ depending on your smartphone's operating system or SmartLog version.

Step 3 Entering the PIN number

- 1 Meter/Smartphone:** The Bluetooth symbol and 6-digit PIN number will appear on the meter's screen.



- 2 Smartphone:** Enter the PIN number into the SmartLog mobile app and touch 'OK'. Make sure the PIN you enter on your smartphone matches the PIN on your meter screen.

③ **Meter:** When your meter and smartphone are paired and connected, the meter will display 'SYNC' and the saved test results will be transferred to your smartphone.



④ **Meter:** When the data transfer is finished, the meter will display 'End' on the screen and it will be automatically turned off after 3 seconds. If the data transfer is failed, the meter will display 'FAIL' and it will be automatically turned off after 3 seconds. Then, repeat Step 2 to 3 to connect your meter with smartphone.



Note

Some smartphones, especially those that are not tested or approved by i-SENS, may be incompatible with your meter. Visit www.i-sens.com/smartlog for more information about supported smartphones.

Adjusting the Date and Time

Step 4 Setting the Year

Press the ◀ or ▶ button to adjust until the correct year appears. When the present year appears, press the **S** button to confirm your selection and to go to the next step.



Step 5 Setting the Month

A number indicating the month will blink on the screen.

Press the ◀ or ▶ button until the correct month appears. Press the **S** button to confirm your selection and to go to the next step.



Step 6 Setting the Date

Press the ◀ or ▶ button until the screen displays the correct date. Press the **S** button to confirm the date and to go to the next step.



Step 7 Setting the Time Format


The meter can be set in the AM/PM 12-hour or the 24-hour clock format.

Press the ◀ or ▶ button to select a format. The AM·PM symbol is not displayed in the 24-hour format. After selecting the format, press the **S** button to go to the next step.



Step 8 Setting the Hour

Press the ◀ or ▶ button until the correct hour appears. After the hour is set, press the **S** button to go to the next step.

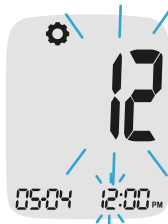

Step 9 Setting the Minute

Press the ◀ or ▶ button until the correct minute appears. After setting the minute, press the **S** button to go to the next step.



Setting the Sound On/OFF

Step 10

On pressing the ◀ or ▶ button, the screen will display 'On' or 'OFF'. Press the **S** button to confirm the selection.


The meter will beep in the following instances if set to On:

- When you push a button to turn on the meter,
- When the test strip is inserted in the meter,
- When the blood sample is absorbed into the test strip and the test starts,
- When the test result is displayed,
- When you press and hold the ◀ button to set the post-meal (PP2) alert,
- When it is time for a pre-set blood glucose test.



If the sound is set to OFF, none of the sound functions will work. After setting the sound, press the **S** button to go to the next step.

Note

The  symbol is displayed only when the sound is set to OFF.


Turning on the Strip Expiration Date Indicator

Step 11

This setting allows you to turn the strip expiration date indicator 'On' or 'OFF'. This setting turns the function 'On' or 'OFF' only. See page 32 to set the strip expiration date.


When 'EP' appears on the screen, press the ◀ or ▶ button. The screen will display 'On' or 'OFF'. Press the **S** button to confirm the setting.

If you do not want to set the indicator, press the **S** button while the screen displays 'OFF'.

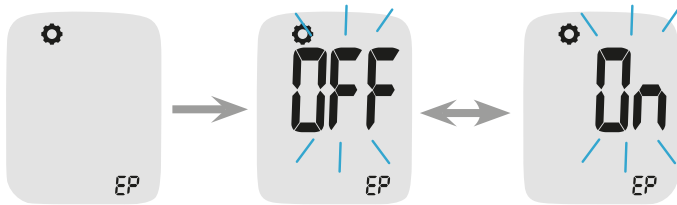

Note


If the pre-set expiration date expires, the meter will display 'EP' when the test strip is inserted. 'EP' shows alternately also when the test result is displayed right after the test. If the expiration date is set to October of 2026, the meter will display 'EP' at the start of November, 2026.

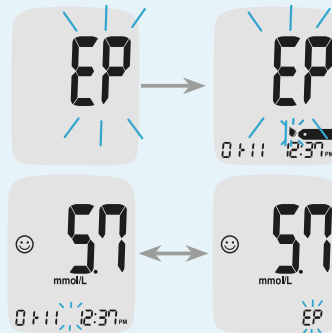

Setting the Hypoglycemia (Lo) Indicator

Step 12

This setting allows you to select the desired level for the hypoglycemia indicator (possible low blood sugar). You will be alerted any time your test result is lower than the selected level. Press the ◀ or ▶ button until the desired hypoglycemia level between 1.1 and 5.0 mmol/L appears. Then, press the **S** button to confirm the level and to go to the next step.

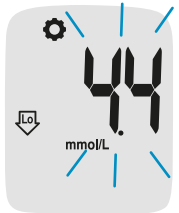

Note

If the test result is lower than the pre-set hypoglycemia level, the meter will display the following.




Caution

Ask your healthcare professional to help you decide what your hypoglycemia level is before setting your level.

Setting the Hyperglycemia (HI) Indicator

Step 13

This setting allows you to select the desired level for the hyperglycemia indicator (possible high blood sugar). You will be alerted any time your test result is higher than the selected level. Press the ◀ or ▶ button until the desired hyperglycemia level between 6.7–19.4 mmol/L appears. Press and hold the **S** button to confirm the hyperglycemia level and turn the meter off.

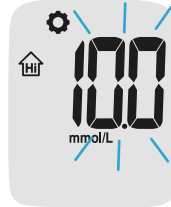

Note

If the test result is greater than the pre-set hyperglycemia level, the meter will display 'Hi'.

If the test result is than between 13.4 and 33.3 mmol/L, 'KET' will blink three times on the screen.



Note

- If your blood glucose is measured to be more than 13.3 mmol/L, you may also have ketones.
- The CareSens S Fit BT meter is not intended to detect ketones.
- If you see 'KET' displayed on your meter, ask your healthcare professional when and how you should test for ketones.

Caution

Ask your healthcare professional to help you decide what your hyperglycemia level is before setting your level.

Note

If the test result is within the selected normal blood glucose range, the smile symbol will be displayed as shown.



Setting the Strip Expiration Date Indicator

Step 1 Entering the Expiration Date Setting

Press and hold the ◀ and ▶ buttons at the same time for 3 seconds to enter the expiration date settings. After all segments flash across the screen, 'EP' will show up.


Note

The strip expiration date is printed on the test strip vial.

Step 2 Setting the Expiration Year

A number indicating the year will blink in the left corner of the screen. Press the ◀ or ▶ button until the correct year appears.


Press the **S** button to confirm the year and set the month.

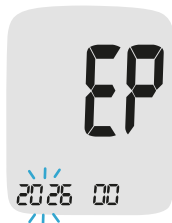

Step 3 Setting the Expiration Month

A number indicating the month will blink at the bottom of the screen. Press the ◀ or ▶ button until the correct month appears.

Press and hold the **S** button for 3 seconds to confirm the month and turn off the meter.

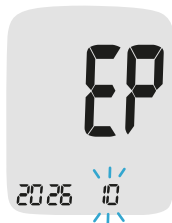

Checking the System



You may check your meter and test strips using the CareSens S Glucose Control Solution (control A and/or B). The CareSens S Glucose Control Solution contains a known amount of glucose and is used to check that the meter and the test strips are working properly. The test strip vials have CareSens S Glucose Control Solution ranges printed on their labels. Compare the result displayed on the meter to the CareSens S Glucose Control Solution range printed on the test strip vial. Before using a new meter or a new vial of test strips, you may conduct a control solution test following the procedure on pages 34–36.

Note


- Use only the CareSens S Glucose Control Solution (available for purchase separately).
- Check the expiration date printed on the bottle. When you first open a control solution bottle, record the discard date (date opened plus three (3) months) in the space provided on the label.
- Make sure your meter, test strips, and control solution are at room temperature before testing. Control solution tests must be done at room temperature 20–25 °C.
- Before using the control solution, shake the bottle, discard the first few drops and wipe the tip clean.
- Close the control solution bottle tightly and store at a temperature between 8–30 °C.


You may do a control solution test:

- When you want to practice the test procedure using the control solution instead of blood,
- When using the meter for the first time,
- Whenever you open a new vial of test strips,
- If the meter or test strips do not function properly,
- If your symptoms are inconsistent with the blood glucose test results and you feel that the meter or test strips are not working properly,
- If you drop or damage the meter.



Control Solution Testing

Step 1 Inserting Test Strip

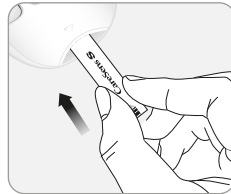Insert a test strip into the meter's test strip port, with the contact bars facing upwards. Gently push the test strip into the port until the meter beeps. Be careful not to bend the strip while pushing it in. The  symbol will be displayed on the screen.




Step 2 Activating Control Solution Test Mode


Press and hold the  button for 3 seconds to activate the Control Solution Test Mode. This will also flag the control solution test result. To undo the control solution flag, press and hold the  button for another 3 seconds.




Step 3 Applying Control Solution to Test Strip

Shake the bottle before each test. Remove the cap and squeeze the bottle to discard the first drop. Then wipe the tip with a clean tissue or cloth.

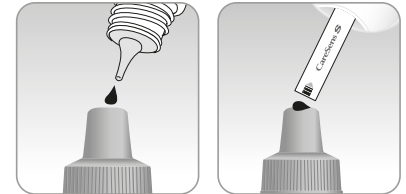

Dispense a drop of control solution onto a clean non-absorbent surface. It helps to squeeze a drop onto the top of the cap as shown. After the  symbol appears on the display, apply the solution to the narrow edge of the test strip until the meter beeps. Make sure the confirmation window fills completely.

Note

The meter may switch off if the control solution sample is not applied within 2 minutes of the  symbol appearing on the screen. If the meter turns off, remove the strip, reinsert, and start from step 1.


Step 4 Waiting for the Result

The display segments will rotate clockwise and a test result will appear after the meter counts down from 6 to 1. The test result with control solution flag is stored in the memory but not included in the averages.



Step 5 Comparing the Result

Compare the result displayed on the meter to the range printed on the test strip vial. The result should fall within the range.

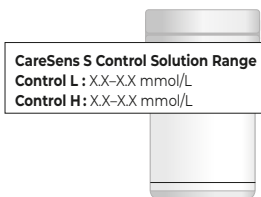

⚠ Caution

The range printed on the test strip vial is for the CareSens S Glucose Control Solution only. It has nothing to do with your blood glucose level.

🔍 Note

The CareSens S Glucose Control Solution can be purchased separately. Please contact Spirit Health on 0800 881 5423.

Comparing the Control Solution Test Results


The test result of each control solution should be within the range printed on the label of the test strip vial. Repeat the control solution test if the test result falls outside of the range. Out of range results may occur in following situations:

Situations	Do This
<ul style="list-style-type: none">• When the control solution bottle was not shaken well,• When the meter, test strip, or the control solution were exposed to high or low temperatures,• When the first drop of the control solution was not discarded or the tip of the bottle was not wiped clean,• When the meter is not functioning properly.	Repeat the control solution test by referring to the Note on page 33.
<ul style="list-style-type: none">• When the control solution is past the expiration date printed on the bottle,• When the control solution is past its discard date (the date the bottle was opened plus three (3) months),• When the control solution is contaminated.	Discard the used control solution and repeat the test using a new bottle of control solution.

If results continue to fall outside the range printed on the test strip vial, the CareSens S test strip and CareSens S Fit BT meter may not be working properly. Do not use your system and contact Spirit Health 24/7 Freephone Support Line at 0800 881 5423.

Using the Lancing Device

You will need a lancing device in order to collect a blood sample. You may use the lancing device that is included in the CareSens S Fit BT Blood Glucose Monitoring System or any other medically approved lancing device.

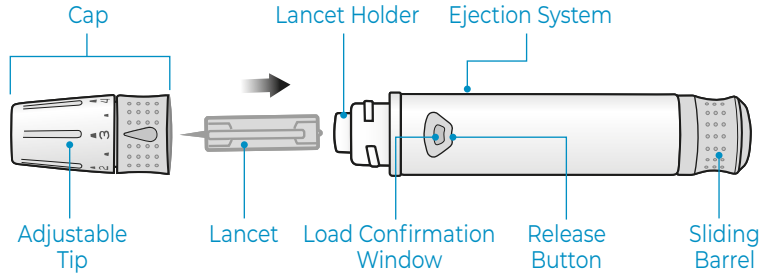

- The lancing device is for use by a single user only and should not be shared with anyone.
- Use a soft cloth or tissue to wipe the lancing device. If necessary, a small amount of alcohol on a soft cloth or tissue may be used.

⚠ Caution

To avoid infection when drawing a sample, do not use a lancet more than once, and:

- Do not use a lancet that has been used by others.
- Always use a new sterile lancet.
- Keep the lancing device clean.


🔍 Note

Repeated puncturing at the same sample site may cause pain or skin calluses (thick hard skin). Choose a different site each time you test.

Preparing the Lancing Device



Step 1

Wash hands and sample site with soap and warm water. Rinse and dry thoroughly.

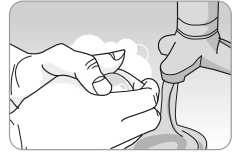


Step 2

Unscrew and remove the lancing device tip.

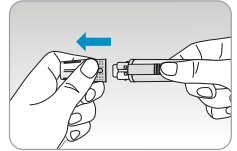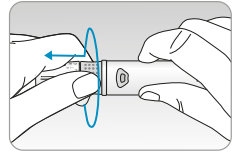



Step 3

Firmly insert a new lancet into the lancet holder. Hold the lancet firmly. Gently twist to pull off protective disk. Save disk to recap lancet after use. Replace lancing device tip.

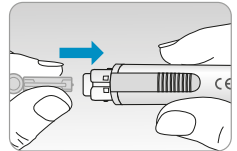

Step 4


Turn the adjustable tip until it is aligned with the load confirmation window and release button as shown.

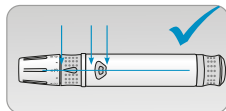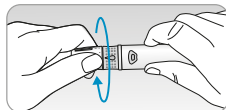

Step 5

The lancing device has six puncture depth settings, numbered 0 through 5 (0 for a shallow puncture, 5 for a deeper puncture).

Choose a depth by rotating the top portion of the adjustable tip until the desired number aligns with the arrow.

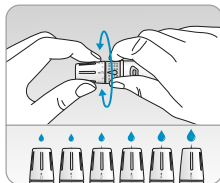


Note

0 = least penetration of lancet into the skin.

5 = most penetration of lancet into the skin.

Step 6

To cock the lancing device, hold the body of lancing device in one hand and pull the sliding barrel with the other hand. The device is loaded when you feel a click and the load confirmation window turns red.

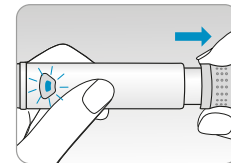


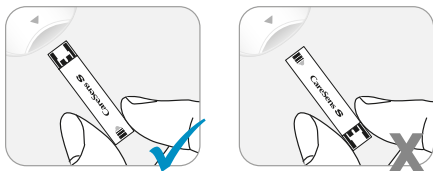
Note

The skin depth to get blood samples will vary for various people at different sample sites. The lancing device's adjustable tip allows the best depth of skin penetration to get an adequate sample size.

Preparing the Meter and Test Strip


Step 7


Insert a test strip with the contact bars facing upwards into the meter's test strip port. Push the strip in gently until the meter beeps. Be careful not to bend the test strip. The  symbol will appear on the screen.




Applying Blood Sample


Step 8

Obtain a blood sample using the lancing device. Place the device against the pad of the finger. Press the release button. Remove the device from the finger. Wait a few seconds for a blood drop to form. A minimum volume of 0.5 microliter is needed to fill the confirmation window (actual size of 0.5 μ L: ).

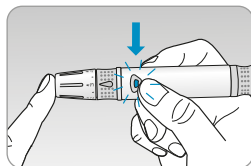

Step 9

After the  symbol appears on the screen, apply the blood sample to the narrow end of the test strip till the meter beeps. At this time, the display segments will rotate clockwise while the blood is going in. If the confirmation window is not filled in time because of abnormal viscosity (thickness and stickiness) or insufficient volume, the **Er4** message may appear. It is recommended to place the test strip vertically into the blood sample site as shown below.

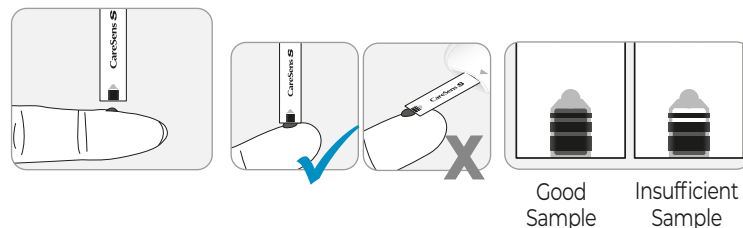

Caution

Do not apply blood on top surface of the test strip.

Caution


Do not allow any foreign substances, such as dirt, blood, or water, enter into the meter. The meter may be damaged or may malfunction. Follow the warning information provided below to prevent possible damage to the meter.

- Do not apply the blood sample directly to the test strip port.

⚠ Caution

- Do not apply the blood sample to the test strip while holding the meter in a way that the tip of the test strip faces upwards. The blood sample may run down the surface of the test strip and flow into the test strip port.
- Do not store your meter in unsanitary or contaminated sites.

🔍 Note

The meter may switch off if the blood sample is not applied within 2 minutes of the  symbol appearing on the screen. If the meter turns off, remove the strip and reinsert it, and start from Step 2.

Step 10


The test result will appear after the meter counts down from 6 to 1. The result will be automatically stored in the meter's memory. If the test strip is removed after the test result is displayed, the meter will automatically switch off after 3 seconds. Discard used test strips safely in disposable containers.



🔍 Note

To transmit glucose data using the Bluetooth feature,

- The Bluetooth feature on the meter must be turned on,
- The meter and a smartphone must be paired,
- The SmartLog mobile app must be launched.

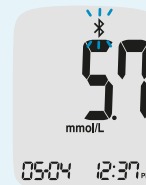


The meter will transmit data in the following cases,

- When the strip is ejected after measuring,
- When the meter is turned on (only when untransmitted data exists).

Step 11

You can attach a flag to a result to indicate particular situations while the strip is still in the meter. When the result is displayed right after a test, press the ◀ or ▶ button to select a pre-meal flag (🍏), a post-meal flag (🍷), or a fasting flag (🚫). When you remove the test strip while the desired flag is blinking, the test result is stored with the flag.


If you do not want to add any flags on the test result, remove the strip after the test result is displayed.

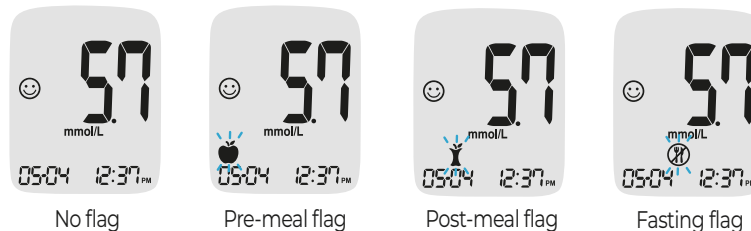

Discarding Used Lancets

Step 1



Unscrew the lancing device tip.

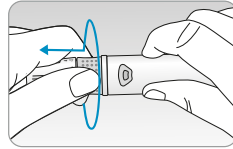

Step 2

Stick the lancet into the saved protective disk.

Push the lancet ejector forward with the thumb to dispose of the used lancet in a proper biohazard container.

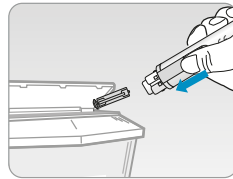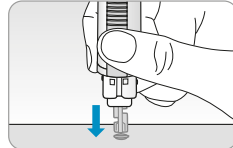

Caution

The lancet is for single use only. Never share or reuse a lancet. Always dispose of lancets properly.

HI and Lo Messages

HI Message

The meter displays results between 1.1–33.3 mmol/L. 'HI' appears when the blood glucose level is greater than 33.3 mmol/L and indicate severe hyperglycemia (much higher than normal glucose levels).

If 'HI' is displayed again upon retesting, please contact your healthcare professional immediately.



Lo Message

'Lo' appears when a test result is less than 1.1 mmol/L and indicates severe hypoglycemia (very low glucose levels).

If 'Lo' is displayed again upon retesting, please contact your healthcare professional immediately.



Note

Please contact the Spirit Health 24/7 Freephone Support Line at 0800 881 5423 or go to www.spirit-health.com, if such messages are displayed even if you do not have hyperglycaemia or hypoglycaemia.

Target Blood Glucose Ranges

Reminders	<i>Your target ranges from your healthcare professional</i>
Time of day	
Before breakfast	
Before lunch or dinner	
1 hour after meals	
2 hours after meals	
Between 2 a.m. and 4 a.m.	

Expected Values

Normal blood glucose levels for an adult without diabetes are below 5.5 mmol/L before meals and fasting* and are less than 7.8 mmol/L two hours after meals.¹


*Fasting is defined as no caloric intake for at least eight hours.

Reference

1. American Diabetes Association (Standards of Medical Care in Diabetes – 2021. *Diabetes Care*), January 2021, vol. 44 (Supplement 1): S15-S33.

Transferring Test Results Using Cable

Test results stored in the CareSens S Fit BT meter can be transferred from the meter to a computer using SmartLog software and cable. The meter screen displays 'PC' when it is connected to the computer using the data cable. For more information, contact the Spirit Health 24/7 Freephone Support Line at 0800 881 5423 or go to www.spirit-health.com.

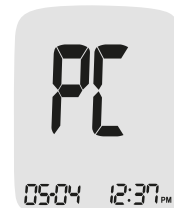

Note

Test results can also be transferred wirelessly using Bluetooth. To pair the meter and your smartphone, see page 22.

Meter Memory


The CareSens S Fit BT meter can save up to 1,000 glucose test results with time and date. If the memory is full, the oldest test result will be deleted and the latest test result will be stored. The meter calculates and displays the averages of total test results, Pre-meal (🍏) test results, Post-meal test (🍷), and Fasting test results (🚫) from the last 1, 7, 14, 30, and 90 days.

Viewing Averages Stored in Memory

Step 1


Press the ◀, ▶ or S button to turn the meter on. The current date and time will be displayed at the bottom of the screen followed by the 1 day average value and the number of the test results saved within the current day.

The number of tests within the current day

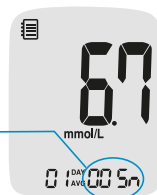

Step 2 Viewing Averages

Press the ◀ button to view the average values of 7, 14, 30 and 90 days and the number of tests performed. (For the average values of Pre-meal, Post-meal or fasting, each representative symbol will appear on the screen with the average values and the number of tests performed.)

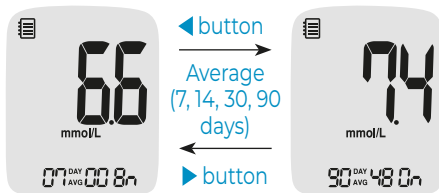

Step 3

Use the ▶ button to scroll back through the averages seen previously. Press the S button to turn off the meter.


Note

The control solution test results saved with the 🧪 symbol are not included in the averages.

Viewing Test Results Stored in Memory


Step 1

Press the ◀, ▶ or S button to turn the meter on. The current date and time will be displayed on the bottom of the screen followed by the 1 day average value and the number of the test results saved within the current day.

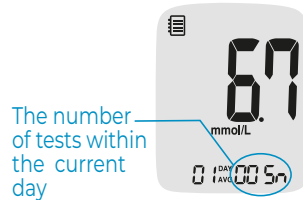



Step 2

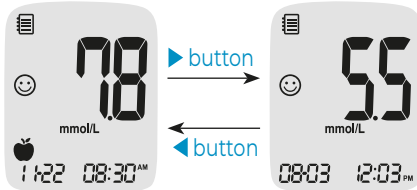Use the ▶ button to scroll through the test results, starting from the most recent and ending with the oldest. Press the ◀ button to return to the result seen previously. After checking the stored test results hold the S button to turn off the meter.



Note

The control solution test results saved with  symbol will be displayed with  symbol when you review the stored test results.

Setting the Alarm Function


Four types of alarms can be set in the CareSens S Fit BT meter: one post-meal alert (PP2 alert) and three time set alarms (alarm 1-3). The PP2 alert goes off 2 hours after setting the alert. The alarms ring for 15 seconds and can be silenced by pressing the ◀, ▶ or S button or by inserting a test strip.

Setting the Post-meal Alert (PP2 alert)

Step 1 Turning the PP2 alert On

Without inserting a test strip, press and hold the ◀ button for 3 seconds to set the post-meal alert.

'PP2', bell (🔔) symbol and 'On' will be displayed. The screen will then automatically change to the memory recall mode. At this time, bell (🔔) symbol, indicating that the PP2 alert has been set, will be displayed on the screen.

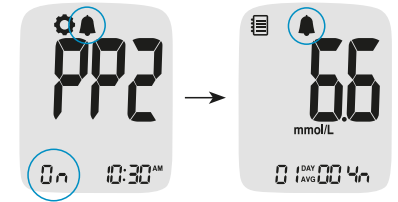


Note

The PP2 alert will automatically turn off if the meter's time setting is adjusted to more than two hours before or just past the currently activated PP2 alert time.

Step 2 Turning the PP2 alert OFF

To turn off the PP2 alert, press and hold the ◀ button for 3 seconds. 'PP2', bell (🔔) symbol and 'OFF' will appear on the screen. Then the screen will change automatically to the memory recall mode without bell (🔔) symbol displayed.

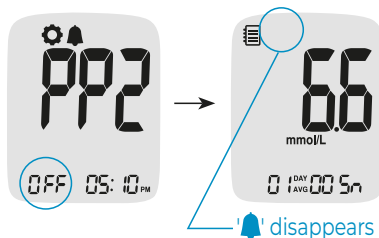


Setting the Time Alarms (alarm 1-3)

Step 1

Without inserting a test strip, press ◀ and the **S** button simultaneously for 3 seconds to enter the time alarm mode. The 'alarm 1' will be displayed while the 'OFF' is blinking on the screen.

On pressing the ▶ button, the 'alarm 1' is set and 'On' is displayed on the screen. Press the ▶ button again to cancel the 'alarm 1'. The 'OFF' will blink on the screen.

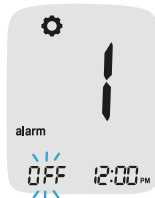

Step 2

Press the ◀ button to set the time of 'alarm 1'. A number representing the hour will blink on the screen. Press the ▶ button to set the hour. On pressing the ◀ button, the number indicating the minute will start blinking. Press the ▶ button to set the minute.



Step 3




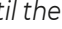

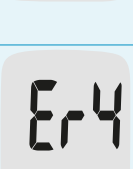
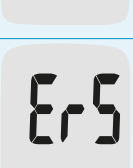
Press the **S** button to finish and to go to 'alarm 2' setting. Repeat steps 2 to 4 to set the remaining alarms time (alarm 2-3).


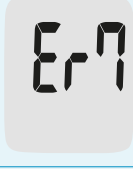
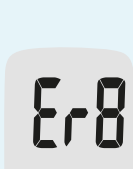


Step 4

Press and hold and hold the **S** button for 3 seconds to finish and turn the meter off.

Understanding Error Messages

	<p>A used test strip was inserted.</p> <ul style="list-style-type: none">› Repeat the test with a new test strip.
	<p>The blood or control solution sample was applied before the  symbol appeared.</p> <ul style="list-style-type: none">› Repeat the test with a new test strip and wait until the  symbol appears before applying the blood or control solution sample.
	<p>The temperature during the test was above or below the operating range.</p> <ul style="list-style-type: none">› Move to an area where the temperature is within the operating range (5–45 °C) and repeat the test after the meter and test strips have reached a temperature within the operating range.
	<p>The blood sample has abnormally high viscosity or insufficient volume.</p> <ul style="list-style-type: none">› Repeat the test with a new test strip.
	<p>This error message may appear when the wrong blood glucose test strip is used instead of CareSens S blood glucose test strip.</p> <ul style="list-style-type: none">› Repeat the test with a CareSens S test strip.

	<p>There is a problem with the meter.</p> <ul style="list-style-type: none">› Contact the Spirit Health 24/7 Freephone Support Line at 0800 881 5423 or go to www.spirit-health.com.
	<p>There is a problem with Bluetooth communication</p> <ul style="list-style-type: none">› Contact the Spirit Health 24/7 Freephone Support Line at 0800 881 5423 or go to www.spirit-health.com.
	<p>An electronic error occurred during the test, commonly caused by foreign materials in the meter port (e.g., blood, dirt, dust), and/or physical damage to the meter or strips due to incorrect use or storage.</p> <ul style="list-style-type: none">› Repeat the test with a new test strip. If the error message persists, contact the Spirit Health 24/7 Freephone Support Line at 0800 881 5423 or go to www.spirit-health.com.

Note

If the error messages persist, please contact the Spirit Health 24/7 Freephone Support Line at 0800 881 5423 or go to www.spirit-health.com.

General Troubleshooting

Problem	Troubleshooting
The display is blank even after inserting a test strip.	<ul style="list-style-type: none"> • Check whether the test strip is inserted with the contact bars facing up. Check if the strip has been inserted completely into the test strip port. • Check if the appropriate test strip was used. • Check whether the battery is inserted with the + side facing up. • Replace the battery.
The test does not start even after applying the blood sample on the strip.	<ul style="list-style-type: none"> • Check if the confirmation window is filled completely. • Repeat the test with a new test strip.
The test result does not match the way you feel.	<ul style="list-style-type: none"> • Repeat the test with a new test strip. • Check the expiration date of the test strip. • Perform control solution test.



Note

If the problem is not resolved, please contact the Spirit Health 24/7 Freephone Support Line at 0800 881 5423 or go to www.spirit-health.com.

Performance Characteristics

The performance of CareSens S Fit BT Blood Glucose Monitoring System has been evaluated in laboratory and in clinical tests.

Accuracy: The accuracy of the CareSens S Fit BT Blood Glucose Monitoring System (Model: GM01NBE) was assessed by comparing blood glucose results obtained by patients with those obtained using a YSI Model 2300 Glucose Analyzer, a laboratory instrument. The following results were obtained by diabetic patients at clinic centers.

Slope	0.998
Y-intercept	0.130 mmol/L
Correlation coefficient (r)	0.9962
Number of samples	600
Range tested	1.5–24.8 mmol/L

System accuracy results for glucose concentration < 5.55 mmol/L

Within ±0.28 mmol/L	Within ±0.56 mmol/L	Within ±0.83 mmol/L
118/180 (65.6 %)	172/180 (95.6 %)	180/180 (100 %)

System accuracy results for glucose concentration ≥ 5.55 mmol/L

Within ±5 %	Within ±10 %	Within ±15 %
298/420 (71.0 %)	403/420 (96.0 %)	417/420 (99.3 %)

System accuracy results for glucose concentrations between 1.5 mmol/L and 24.8 mmol/L

Within ± 0.83 mmol/L and Within ± 15 %
597/600 (99.5 %)

Precision: The precision studies were performed in a laboratory using CareSens S Fit BT Blood Glucose Monitoring Systems.

<i>Within Run Precision</i>		
Blood average	2.0 mmol/L	SD = 0.1 mmol/L
	3.8 mmol/L	SD = 0.1 mmol/L
	7.5 mmol/L	CV = 3.0 %
	11.8 mmol/L	CV = 2.6 %
	16.5 mmol/L	CV = 3.0 %

<i>Between Run Precision</i>		
Control solution average	2.3 mmol/L	SD = 0.1 mmol/L
	7.2 mmol/L	CV = 2.3 %
	19.7 mmol/L	CV = 3.7 %

This study shows that there could be variation of up to 3.7 %.

Influence Quantities

Packed Cell Volume (Hematocrit)

Packed cell volume evaluation was conducted in various hematocrit levels. The range of hematocrit levels within the acceptance criteria is 15–65 %.

Interferences

The effect of various interfering substances was evaluated in whole blood samples. The presence of the following substances within the given concentrations does not affect blood glucose measurements. Higher concentrations of the substances shown below may cause inaccurate blood glucose results.

No.	Interferent	Concentration
1	Acetaminophen (paracetamol)	1.32 mmol/L
2	Ascorbic acid	0.17 mmol/L
3	Bilirubin (Unconjugated)	0.34 mmol/L
4	Cholesterol	12.93 mmol/L
5	Creatinine	2.65 mmol/L
6	Dopamine	0.85 mmol/L
7	EDTA	6.16 mmol/L
8	Galactose	3.33 mmol/L
9	Gentisic acid	3.24 mmol/L
10	Glutathione (Reduced)	2.99 mmol/L
11	Hemoglobin	0.31 mmol/L

Warranty Information

Manufacturer's Warranty

i-SENS, Inc. warrants that the CareSens S Fit BT meter shall be free of defects in material and workmanship in normal use for a period of five (5) years. The meter must have been subjected to normal use. The warranty does not cover improper handling, tampering, use, or service of the meter. Any claim must be made within the warranty period.

i-SENS will, at its discretion, repair or replace a defective meter or meter part that is covered by this warranty. As a matter of warranty policy, i-SENS will not reimburse the consumer's purchase price.

Obtaining Warranty Service

To obtain warranty service, you must return the defective meter or meter part along with proof of purchase to your nearest i-SENS sales or customer service representative.

No.	Interferent	Concentration
12	Heparin	8000 U/dL
13	Ibuprofen	2.42 mmol/L
14	Icodextrin	1094 mg/dL
15	L-Dopa (L-3,4-dihydroxyphenylalanine)	0.25 mmol/L
16	Maltose	29.21 mmol/L
17	Methyl-DOPA	0.07 mmol/L
18	Pralidoxime Iodide (PAM)	0.95 mmol/L
19	Sodium Salicylate	5.07 mmol/L
20	Tolbutamide	3.70 mmol/L
21	Tolazamide	3.21 mmol/L
22	Triglycerides	37.26 mmol/L
23	Uric acid	1.49 mmol/L
24	Xylose	19.98 mmol/L

User Performance Evaluation

A study evaluating glucose values from fingertip capillary blood samples obtained by 100 lay persons showed the following results:

100 % within ± 0.83 mmol/L of the medical laboratory values at glucose concentrations below 5.55 mmol/L, and 97.2 % within ± 15 % of the medical laboratory values at glucose concentrations at or above 5.55 mmol/L.

MEMO

MEMO

MEMO

MEMO



Visit the web page below
to see supported devices/
download app (scan the QR
code for easy access).

www.i-sens.com/smartlog

i-sens

spirit
health

Spirit Healthcare Ltd.

Spirit House, Saffron Way,
Off Saffron Lane, Leicester, LE2 6UP
Customer Support Tel : 0800 881 5423

www.spirit-health.com

 24/7 NON-AUTOMATED FREE SUPPORT
0800 881 5423

 **i-SENS, Inc.**
43, Banpo-daero 28-gil
Seocho-gu, Seoul 06646, Korea
www.i-sens.com

EC REP **Medical Technology Promedt
Consulting GmbH**
Ernst-Heckel-Straße 7
66386 St. Ingbert, Germany

© 2024 i-SENS, Inc. All Rights Reserved.
PGUAA-0000602 REV0 2024-10

